



Bovine Pericardium Patch

 **BRAILE**

LIFE-SAVING TECHNOLOGIES

The Bovine Pericardium Patch is composed of bovine pericardium, previously treated with glutaraldehyde solution and preserved in 4% formaldehyde solution.


This chemical treatment provides it with appropriate properties of resistance, flexibility and absence of antigenicity.

Indications for Use



Cardiovascular Surgery

- Closure of intercavitary defects;
- Aortic annulus enlargement in valve replacements;
- Atrial and ventricular reconstruction;
- Coronary cerclage in procedures without extracorporeal circulation and surgical corrections of aortic conditions;
- Peripheral arterioplasties;
- Expansion of the RV outflow tract;
- Treatment of aortic and left ventricle aneurysms;
- Mustard procedure;
- Reconstruction of the right ventricle and pulmonary artery outflow tract.

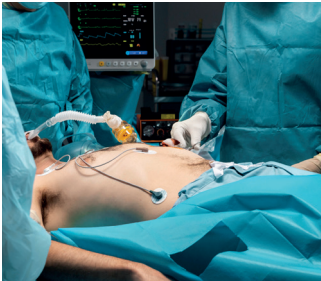

Thoracic Surgery

- Bronchial stump closure;
- Diaphragmatic hernia closure;
- Treatment of chest wall defects (congenital and traumatic).



General Surgery

- Surgical treatment of hernias and eventrations.



Urologic Surgery

- Cytoplasties;
- Ureteroplasties.



Neurosurgery

- Dura mater substitution.

Additional Information

- Single-use product.
- The patch should be kept between 10 and 30 °C and should never be frozen or submitted to temperatures over 40 °C.



Code	Model
4293	Bovine Pericardium Patch: 11 x 6 cm
4248	Bovine Pericardium Patch: 11 x 9 cm

Instructions for Use

- Take the patch from the recipient using an atraumatic clamp and should handle it only with the hands to avoid any traumatism.
- The product implantation technique is followed by the medical team discretion.
- Thoroughly wash the pericardium before usage in a recipient containing approximately 500 mL (17.6 fl oz) of saline, renewing the solution at least three times within 15 minutes.

Date of expiry: Two years from the manufacturing date. It is considered to be sterile as long as the seal is not broken.

Follow us on social media:

 /brailebiomedica

 @brailebiomedica



Braile Biomédica Indústria, Comércio e Representações Ltda.

Av. Pres. Juscelino K. de Oliveira, 1505 | Jardim Tarraf I

CEP 15091-450 | São José do Rio Preto - SP | Brasil

CNPJ: 52.828.936/0001-09 | Anvisa no. 10159030049

Technical manager: Vlademir D. A. Ramirez - CRF-SP 09010